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What is the CaroskinTM Lymph Care concept?

The CaroskinTM Lymph Care concept consists of different CaroskinTM products which have been put together to guarantee you the best possible lymphoedema compression treatment using top quality products.

These products are:

- CaroskinTM Gloves & Arm sleeves:
 - Flat knitting **seamless**
 - S-Line Flex: Made to measure
 - S-Line Flex: Standard
 - Flat knitting **with seam**
 - Active: Made to measure
 - Active: Standard
 - Strong: Made to measure

Why, when, and how are CaroskinTM Comfort Lymph Care compression garments worn?

The standardised range of CaroskinTM Comfort Lymph Care compression garments is an extensive range of fully fledged compression class II compression garments which are designed to prevent or treat lymphoedema in the arms or hands by means of constant, sustainable, and therapeutic compression.

CaroskinTM Comfort Lymph Care compression garments have the following features:

- Made to measure or standard
- Therapeutic compression function (compression class II).
- A range which takes account of the different circumferences of the hands or arms.
- An open finger glove can replace a closed finger glove to give more comfort through free finger tips.
- Is anti-allergic – consists of natural polymers.
- Half knitting at the ends of the sleeve = not as thick: easy to combine with a glove.
- S-Line: soft and comfortable elasticity ensures plenty of comfort due to the seamless knitting technique.
- Active quality: Advantages of the natural crustacean extracts:
 - Lightly hydrating
 - Heals small wounds - styptic
 - Bacteriostatic features
 - High transpiration absorption

By wearing the compression garments in the prescribed manner and maintaining them in the correct manner, you will achieve the desired results.

Practical tips:

- Always keep your skin clean and dry.
- Remove rings, watches, bracelets, and/or jewels when putting on CaroskinTM Comfort Lymph Care compression garments.
- Cut any long nails short (to wear a closed glove).
- Take your time when putting the garments on.
- Contact the prescribing doctor if you think repairs may be needed.
- For warranty please keep the original packaging.

Washing and maintenance advice

- Washing CaroskinTM Comfort Lymph Care compression garments frequently (e.g. every 2 days) at 30°C using an ecological detergent extends their lifespan.
- Wash dark colours separately.
- Rinse frequently.
- Do not wring and do not dry near a heater, in an airing cupboard, or in direct sunlight: pat dry and air-dry.
- The CaroskinTM Pressure Garments retain their therapeutic pressure value for a minimum period of 2-3 months (if used and maintained correctly). The therapeutic pressure will last longer if you are able to switch between several identical pressure garments. However, re-measurement of the patient should be considered in the event of rapid increase or decrease of circumference.

Warning

- Protect your arms or hands against direct sunlight or rapid temperature changes (e.g. from Turkish baths, saunas, sunbeds etc.).
- Observe the rules of good hygiene.
- Only use pH neutral products for the maintenance of your Lymph Care compression garments. Otherwise the fibres may be affected and this can reduce the pressure.
- Since lymphoedema is a very complicated subject and each patient presents a separate case, the doctor who treats you will prescribe personalised treatment.

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